INDICAÇÃO:

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INDICATION:

Indicated for implant prosthesis, partial denture support connector, Inlay, Onlay, Crown & bridge joint connector, production of inside and outside of conus crowns.

CONTRAINDICATIONS:

Keep out of reach of children and the elderly.

WARNINGS/PRECAUTIONS:

- For dental use only.
- Keep the container closed after use.
- To prevent the resin from light-curing on the metal dosing nozzle, store it in a place protected from light.
- Store away from light and heat. Keep in a dry environment at room temperature.
- Disposal must be carried out in accordance with local regulations.

INSTRUCIONS FOR USE:

First, vaseline or a separating agente should be applied to the surface of the dentures. Then, remove the protective cover from the syringe, placing the needles in it. Slightly press the piston to shape de material. The shape is adjusted using the usual method according to the purpose. Polymerize for about 1 min using a light curing device with the light spectrum at 350nm to 480nm.

CHARACTERISTICS/BENEFITS:

- Odorless
- Easy and quick to shape
- Fast curing
- Viscous consistency (thixotropic), like gel
- Comes in the ideal consistency for immediate use
- Curing in thin layers or continuously
- Flexibility, agility in preparing models for casting

PRESENTATIONS:

1 syringe with 12ml and 3 measuring nozzles

SHADE:

Red

COMPOSITION:

- Bisphenol A Diglycidyl Ether Dimethacrylate
- Hydrophobic Fumed Silica
- Methylamino Anthraquinone

ATTENTION: Use our free support channels to obtain the correlation between the purchased product and the corresponding usage instruction version. To obtain, free of charge, these Instructions for Use in printed format, ask the customer service (SAC) on 0800 058 8888.

SHELF LIFE:

2 years after the date of manufacture

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