INDICAÇÃO:

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INDICATIONS

Indicated for sculpting characterized gums, relieving in plaster models, fixing and mounting teeth.

ADVERTISEMENTS/PRECAUTIONS:

Store away from light and heat. Keep in a cool, dry environment.

Use clean instruments to avoid contamination or impurities when handling the wax.

INSTRUCTIONS FOR USE:

Evoden's Create Wax have been designed to provide excellent performance in gingival characterization work, fixing and mounting teeth and relief in plaster models. Follow the instructions below to ensure the best performance and professional results:

Material Preparation

- Make sure the plaster model is clean and dry before starting to apply the wax.
- Choose the most appropriate shade of wax for the work to be carried out (gums, fixing teeth, or relief).

Heating the wax

- Heat the Create Wax using an electric heater or heated spatula until it reaches a moldable consistency suitable for carving.
- Avoid overheating to maintain the wax's properties and ensure a smooth application.

Gum Characterization Application

- Use a spatula to apply the wax to the desired area of the model.
- Shape the wax to simulate the contour and natural texture of the gums.
- Make adjustments as necessary, sculpting fine details to achieve a realistic and aesthetic finish.

Fixing and mounting teeth

- To attach temporary teeth to dentures or bridges, ensure that both the teeth and the plaster model are clean.
- Heat the wax until it becomes malleable and apply a small amount to the base of the tooth.
- Position the tooth in the correct place and press lightly to ensure a good grip.
- The wax should provide temporary fixation, allowing for removal and adjustments until the final prosthesis is completed.

Relieving Plaster Models

- When using wax for pressure relief on specific areas of plaster models, apply a thin layer of Create Wax to the desired areas.
- The wax will act as a separator, relieving pressure and ensuring greater comfort when fitting the prostheses.

Finishing

- After shaping and adjusting the wax, make a final inspection to ensure that all the details are as desired.

- If necessary, use a heated spatula to smooth the surface and ensure an even finish.

Removal and correction

- To make corrections or remove the wax, heat it slightly and carefully remove it with a spatula without damaging the model or temporary teeth.

Evoden's Create Wax guarantee flexibility and precision for dental professionals, contributing to high-quality aesthetic and functional results.

FEATURES/BENEFITS:

• Malleable texture and easy to shape

Differential: Create Wax has an ideal consistency, making it easy to handle when sculpting fine details, such as gingival textures and dental contours.

Benefit: Saves time and effort, with greater precision in detail, allowing for more personalized, high-quality results.

• High Thermal Stability

Differential: Unlike many common waxes, Create Wax maintains its physical properties even under different temperature conditions, not deforming or softening excessively.

Benefit: Ensures greater control during work, avoiding the need for frequent corrections and allowing for a more precise and consistent application.

• Versatile application

Differential: Create Wax are indicated both for the aesthetic characterization of the gums and for the temporary fixation of provisional teeth and for reliefs in plaster models.

Benefit: A single wax can meet several needs in the prosthetic process, simplifying work and reducing the amount of materials required.

• Sophisticated aesthetic finish

Differential: Create Wax allows the creation of smooth surfaces and refined aesthetic details, such as natural color and shape transitions, perfectly mimicking the appearance of natural gums and teeth.

Benefit: Final results that exceed expectations, with superior aesthetics and greater patient satisfaction.

• Easy to remove and adjust

Differential: If corrections or adjustments need to be made, Create Wax can be easily removed using moderate heat, without compromising the model or tooth structure.

Benefit: Flexibility in the adjustment process, allowing quick and efficient modifications without causing damage to the initial work.

PRESENTATION:

250g bottle

COLORS:

Dentin 62
Medium Pink
Light Pink
Intensive Purple
Intensive Red I

COMPOSITION:

Synthetic polymers, Hydrocarbons, Natural wax and Pigments.

VALIDITY:

10 years.

Product exempt from registration by the Ministry of Health.

TECHNICAL MANAGER

Jéssica Augusto.
CRQ IV-04270957

ATTENTION: Use our free support channels to obtain the correlation between the purchased product and the corresponding usage instruction version. To obtain, free of charge, these Instructions for Use in printed format, ask the customer service (SAC) on 0800 058 8888.

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