INDICAÇÃO:

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INDICATION:

For use in permanent or removable, total or partial dentures, overdentures, crowns or temporary bridges.

CONTRAINDICATIONS:

There are no known adverse side effects, when the product is used following the usual procedures.

WARNINGS AND PRECAUTIONS:

Keep away from light and heat.
For dental use only.
Use the Duodent shade guide and articulation table.

INSTRUCTIONS FOR USE:

Follow the usual procedures for selection, assembly, articulation, acrylization, microbiological control and maintenance of the denture, as well as for individual protection.

For better adhesion of the teeth to the polymer base of the prosthesis, remove the brightness from the base of the tooth with an abrasive tool. After insulation of the flask and before application of the resin, clean the base of the teeth with wax remover.

Recommendations for the patient: To increase durability of the teeth, daily hygiene is recommended without use of aggressive substances, like acids or alkalis.

CHARACTERISTICS / BENEFITS:

- Pressing of 2 layers, providing a natural appearance.
- Produced through CAD-CAM steel moulds, with great symmetry and details.
- European model set compatible with diverse age group and with better anatomical adjustment.
- Manufacture with selected raw materials, compatible to those of the denture base, improving it's adhesion.
- High stability of colors, with more natural colors.
- Biocompatible pigments
- With fluorescence, providing a natural appearance.
- Complies with ISO 22112

MODELS:**Upper Anterior:**

E2, E3, E4, F4, G3, H3, H4, L2, L4, O32, O36, R13, R14, R17, T25, T4B, O31, R11, R12, R66, T21, T22, T44.

Lower Anterior:

K6, K10, K12, L1, L3, L5, L7 e L10.

Upper and Lower Posterior:

M2, M3, M4, M5 e M6.

SHADE:

60,62,65*,66,67,69,77*,81*,
A1, A2, A3, A3.5, A4, B1, B3, C2, C3, D3, BL2, BL3.

COMPOSITION:

Polymethylmethacrylate
Ethylene glycol Dimethacrylate
Fluorescent
Biocompatible Pigments
Organic additive

SHELF LIFE

10 years as of the date of manufacture

ANVISA N°: 82426979002

TECNICAL RESPONSIBLE:

Jéssica Augusto.
CRQ IV-04270957

ARTICULATION CHART		
ANTERIOR		POSTERIORES
E4	L5	M4-M6
G3	K6	M4
O31	L5	M3
O32	L3	M5
O36	L7	M4
V26	L3-L10	M3-M5
L2	K6	M4
L4	K12-K10	M4-M6
R11	L1	M3
R12	L5	M3
R13	L5	M3-M5
R14	L7	M4-M6
R17	K10	M6
R66	L5	M5
V64	K6	M3-M5
E2	L5	M2
E3	K6	M4
F4	L10	M4-M6
H3	L3	M3
H4	L3	M5
T21	L3	M3
T22	L3	M3
T25	L10	M4
T44	L3	M3-M5
T4B	L5	M2
V25	L3-L5	M5
V6B	K10	M6
V2D	L3	M2
V3P	L10	M5

ATTENTION: Use our free support channels to obtain the correlation between the purchased product and the corresponding usage instruction version. To obtain, free of charge, these Instructions for Use in printed format, ask the customer service (SAC) on 0800 058 8888.

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