INDICAÇÃO:

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INDICATION:

WAX block for milling casting or injection patterns (substructures and crowns cemented or screwed), created and milled by CAD/CAM system.

STORAGE:

- The product colors may vary due to the use of natural wax.
- Keep away from heat and light.

USE INSTRUCTIONS:

The EVOBLOCK should be fixed in the milling machine according to the instructions from the device manufacturers.

LIMITS:

- Posteriors: Do consider indications of material to be done, as metal (casting), ceramics pressing, etc.
- Minimum thickness: 1 mm occlusal and cervical 0.6mm
- Minimum Diameter connection: Anteriors: 3.6mm and Posterior: 4.4 mm

Important Note: Avoid overheating during machining and finishing.
Recommended operating temperature: Between 20°C (68°F) to 30°C (86°F).

BENEFITS:

- Reduced milling time;
- Easy extraction;
- Ideal for casting and injection processing
- Special high quality milling wax;
- Good contrast.
- 100% burnout
- Low tool consumption during milling process;

MODELS:

EBW, EBZ e EBA.

COLORS:

gray, ivory, blue and white.

HEIGHTS:

According to the table

Models/ Heights (mm)	13	14	16	20	22	25
EBA	X	-	-	X	-	-
EBW	-	X	X	X	-	-
EBZ	-	X	X	X	-	-

COMPOSITION:

Synthetic hydrocarbon polymers, natural wax, and pigments.

EXPIRATION DATE:

10 years from the manufacturing date.

PACKAGING:

1 piece.

**PRODUCT DECLARED EXEMPT FROM REGISTRATION BY
THE MINISTRY OF HEALTH.****TECHNICAL RESPONSIBLE:**

Jéssica Augusto.
CRQ IV-04270957

ATTENTION: Use our free service channels to obtain the correlation between the supplied product and the version of the corresponding Instructions for Use. To obtain, free of charge, these Instructions for Use in printed form, please ask Customer Service at 0800 058 8888

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