INDICAÇÃO:

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INDICATION:

It is used in the machining technique for making crowns and bridges, as long-lasting, cemented or screw-retained temporaries, machined by CAM System from various manufacturers.

CONTRAINDICATIONS:

Contraindicated for people allergic to any of the components

INSTRUCTION FOR USE:

Attach the block to the machine following the equipment manufacturer's instructions.

CHARACTERISTICS AND BENEFITS:

- Color stability
- Easy polishing
- Fidelity of machined details
- Compatible with all the various machining systems
- High mechanical strength, facilitating chip detachment during the machining process
- Uniform color distribution in the multilayer blocks
- Resistance to bending

MECHANICAL PROPERTIES

| | |
|--|--|
| Modulus of Elasticity | > 2.200 Mpa |
| Bending Strength | 100 MPa |
| Toughness (Charpy): (Izod): | 1,41 KJ / m² 21,65 J / m |
| Tensile Strength | 66,71 MPa |
| Vickers Hardness HV (0,05/10) | 18,7 |
| Water Sorption ISO 10477 | Max 32µg/mm³ |
| Solubility in water | Max 1,6µg/mm³ |
| Residual monomer: | Max 2,2% |

MODELS:

EBW, EBZ, EBA, and EBS40 (multilayer only)

COLORS:

A1, A2, A3, A3,5, A4, B1, B3, C2, C3, D3, BL2, BL3.
Medium Pink B - Light Pink - Dark Pink.

HEIGHTS:

| Models/ Heights (mm) | 13 | 14 | 16 | 20 | 22 | 25 |
|-------------------------|----|----|----|----|----|----|
| EBA | X | - | - | X | - | X |
| EBW | - | X | X | X | X | X |
| EBZ | - | X | X | X | X | X |

OBS: Heights of 25mm only on Evoblock multicolor.

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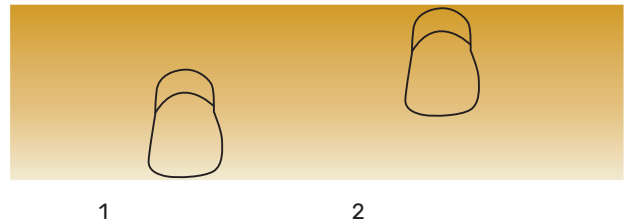
Blocks and Monocolor: Maximum 2 fixed elements plus one dropdown element.

Minimum thickness: 1 mm occlusal and 0.6 mm cervical.

Minimum connection diameters: Anterior: 3.6 mm; Rear: 4.4 mm.

The temporary can be fixed with all cements for temporary fixation. Temporary cements with Eugenol may decrease the effectiveness of resin cements.

Illustrative images of the element positioned at different heights of the block, changing the color of the incisal.

Fig. 1: Piece positioned in the lower region of the block: More incisal.
Fig. 2: Piece positioned in the upper region of the block: Less incisal.

COMPOSITION:

Polymethyl methacrylate, EDMA, Fluorescent, organic additives and Biocompatible Pigments.

EXPIRATION DATE:

10 years after the manufacturing date.

PACKAGING:

1 or 6 pieces.

STORAGE:

keep in a dry place and protected from sunlight.

ANVISA N°:

82426979008

TECHNICAL RESPONSIBLE:

Jéssica Augusto.
CRQ IV-04270957

ATTENTION: Use our free service channels to obtain the correlation between the supplied product and the version of the corresponding Instructions for Use. To obtain, free of charge, these Instructions for Use in printed form, please ask Customer Service at 0800 058 8888

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