INDICAÇÃO:

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INDICATION:

Bivalent polymerizable acrylic resin for elaboration of crowns, bridges, sculptures of provisional dentures, fixation of veneers and jackets, fillings of retentions in RTD, RPD and filling of cavities in acrylic teeth characterization works. To replace part of chewing system of the human body.

CONTRAINDICATIONS:

The use of this medical product must not be used in people with known skin sensitization (contact dermatitis) to polymerizable monomers, as they can cause allergic reactions to hypersensitive people.

INSTRUCIONS FOR USE:

Prior to starting any work, the containers must be completely clean and free of traces of wax or grease.

PREPARATION:

The containers and the hands must be free of contamination to avoid color alterations.

The use of the product in ventilated places and shaking well the bottle before using is recommended.

The works can be executed with brush, spatula or manipulated.

DOSING AND PLASTICIZATION:

The proportion of powder and Liquid must always result in a well fluid mixture.

For complex and time-consuming works, we recommend for several powder / liquid dosages to be made, so that you may always work with a fluid mixture.

The other option is the Powder and Liquid system when a layer of resin is placed and then, with dropper, it is saturated with monomer and immediately a new layer of resin is applied, until the desired thickness is reached in the work.

In this method, a total saturation with monomer over the resin is essential so that it may not dry up and in the previous application, take the work to the polymerizer still in the plastic phase.

So that you may be able to obtain colors and results closest to the scale it is important to observe the right proportion between the Body (Dentin) and the Enamel (Incisor).

Consider that to whiten the crown's hue, the incisor layer is extended along the gingival third. To darken the crown's hue, limit the layer to the medial third, exposing more the body's color. Tone of intermediary colors can be obtained by mixing proportional parts of 2 powders whose colors are closer to the desired tone.

POLYMERIZATION:

Works done with Selfpolymerizable Liquids must be taken to the hydropneumatic polymerizer to reach complete polymerization.

In a normal cycle, polymerize for 20 minutes at a pressure of 20 pounds.

For works with Thermopolymerizable Liquids include the plaster work or silicon wall and bring to an electric hydropneumatic polymerizer for 30 minutes approximately 100°C, with a pressure of 60 pounds.

FINISHING AND POLISHING:

Start finishing with bits and mills, pass through abrasive stones and conclude with fine textured sandpapers, perform polishing with soft brush and polishing rods.

Observations: To assure the best quality in your works, always use monomers Evocor with Crosslink and Fluorescent, because its specific formation provides excellent results.

WARNINGS/PRECAUTIONS:

- For dental use only.
- Liquid: Flammable and non-toxic material (Flash-point: 10°C/50°F).
- Resin: May form flammable air/dust mixtures.
- As with all acrylic resins, it is recommended to handle the product in ventilated areas, using nitrile gloves and protective goggles; however, handling is not recommended for pregnant women, nursing mothers and people with recognized allergies to this type of product.
- May cause skin irritation in susceptible people.
- Keep away from light and heat. Keep in a dry and cool place.
- Do not dispose of waste into sewers and water courses or in domestic waste.
- Products that are not compatible with the resin/monomer: alcohol, water, chloroform, acetone, petroleum jelly and other solvents.
- The bases of the synthetic teeth must be ready in accordance with the Instructions for Use for the correct support of the prosthesis.
- We do not recommend the use of wax solvents. The residual may cause a less connection of the teeth with the acrylic base.
- Recommended mechanical polishing for all types of resin.

CHARACTERISTICS/BENEFITS:

- Rigorous selection and raw materials receipt tests with successive tests during all manufacture cycles, ensure the results specified in the final product.
- It supports the most varied polymerization methods.
- Range of colors, assuring repeatability and customization of your works.

- The fluids with Fluorescent and Crosslink provide durability and natural aesthetics.
- Uniform polymerization in all parts of the denture, even in the thicker parts.
- Ease in application for execution of work, finishing, polishing and brilliance.

PRESENTATIONS:

For another presentations, contact us.

POWDER: 50g
LIQUID: 50ml

SHADES:

60, 62, 65, 66, 67, 69, 77, 81, BL2, INCISAL I e INCISAL II

2A, 2B, 1A, 1D, CERVICAL I, CERVICAL II, INCISAL III

COMPOSITION:

POWDER

Polymethylmethacrylate
Benzoyl peroxide
Biocompatible pigments

LIQUID

SELF-POLYMERIZABLE LIQUID

Methylmethacrylate
EDMA (Crosslink)
Inhibitor
Fluorescent
DMT

THERMO-POLYMERIZABLE LIQUID

Methylmethacrylate
EDMA (Crosslink)
Inhibitor
Fluorescent

SHELF LIFE:

POWDER: 10 years as of the date of manufacture.
LIQUID: 2 years as of date of manufacture.

ANVISA N°:

POWDER: 82426979001
SELF-POLYMERIZABLE LIQUID: 82426979005
THERMO-POLYMERIZABLE LIQUID: 82426979004

TECHNICAL RESPONSIBLE:

Jéssica Augusto
CRQ IV-04270957

***For anoter presentantions, contact us.**

ATTENTION: Use our free support channels to obtain the correlation between the purchased product and the corresponding usage instruction version. To obtain, free of charge, these Instructions for Use in printed format, ask the customer service (SAC) on 0800 058 8888.

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