INDICAÇÃO:

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INDICATION:

For use in permanent or removable, total or partial dentures, overdentures, crowns or temporary bridges.

CONTRAINDICATIONS:

There are no known adverse side effects, when the product is used following the usual procedures.

WARNINGS AND PRECAUTIONS:

Keep away from light and heat.
For dental use only.
Use the Evodent shade guide and articulation table.

INSTRUCTIONS FOR USE:

Follow the usual procedures for selection, assembly, articulation, acrylization, microbiological control and maintenance of the denture, as well as for individual protection.

For better adhesion of the teeth to the polymer base of the prosthesis, remove the brightness from the base of the tooth with an abrasive tool. After insulation of the flask and before application of the resin, clean the base of the teeth with wax remover.

Recommendations for the patient: To increase durability of the teeth, daily hygiene is recommended without use of aggressive substances, like acids or alkalis.

CHARACTERISTICS / BENEFITS:

- Pressing of 2 layers, providing a natural appearance.
- Produced through CAD-CAM steel moulds, with great symmetry and details.
- American model set compatible with diverse age group and with better anatomical adjustment.
- Manufacture with selected raw materials, compatible to those of the denture base, improving it's adhesion.
- High stability of colors, with more natural colors.
- Biocompatible pigments
- With fluorescence, providing a natural appearance.
- Complies with ISO 22112

MODELS:

Upper Anterior:

1D, 2D, 3D, 3M, 2N, 3N, 2P, 3P, A23, A25, A26, 38, 133, 263, 264, 266

Lower Anterior:

2 D, 3 D, 3 M, 2 N, 3 N, 2P, 3P, A 25, A26, 266

Upper and Lower Posterior:

30 M, 32 M, 30 L, 32 L, 34 L.

SHADE:

60, 62, 65*, 66, 67, 69, 77*, 81* – A1, A2, A3, A3.5, A4, B1, B3, C2, C3, D3, BL2, BL3.

COMPOSITION:

Polymethylmethacrylate
Ethylene Glycol Dimethacrylate
Fluorescent
Biocompatible Pigments
Organic additive

SHELF LIFE:

10 years as of the date of manufacture

ANVISA N°: 82426979002

TECHNICAL RESPONSIBLE:

Jéssica Augusto.
CRQ IV-04270957

ARTICULATION CHART	
ANTERIOR	POSTERIORES
2D - 2D	30L - 30M
3D - 3D	30M - 32M
3M - 3M	30M
2N - 2N	30L
3N - 3N	30L - 32M
2P - 2P	32L - 34L
3P - 3P	32L - 32M
A25 - A25	30M - 32M
A26 - A26	32M
263 - 2N	30L - 30M
264 - A25	32L - 32M
266 - 266	34L
38 - 266	34L
1D - 3M	30M
133 - 3M	30M
A23 - 3M	30M

ATTENTION: Use our free support channels to obtain the correlation between the purchased product and the corresponding usage instruction version. To obtain, free of charge, these Instructions for Use in printed format, ask the customer service (SAC) on 0800 058 8888.

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