### INDICAÇÃO:

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### INDICATION:

Evolay Opaque is used for masking metal surfaces and pins in prosthetic work.

### CONTRAINDICATIONS:

The use of this medical product must not be used in people with known skin sensitization (contact dermatitis) to polymerizable monomers, as they can cause allergic reactions to hypersensitive people.

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Shake before use.

Clean the metal well with alcohol and let it dry. When applying, always use a clean brush to apply a layer of Evolay Silane. Let dry for 1 minute before applying Evolay Opaque. Keep it in the oven to dry between 40 and 60 °C for at least 30 minutes.

### WARNINGS/PRECAUTIONS:

- When there is proven allergy to any of the components of the formula, the product should not be used.
- Use the product in a ventilated place, using a spatula, gloves, mask and safety glasses.
- The handling of this product is not recommended for pregnant or lactating women.
- Keep the container closed.
- Do not reuse the packaging.
- Follow the instructions for use correctly, in order to maintain the properties and performance of the product defined by the manufacturer.
- Follow the usual procedures for microbiological control during manipulation.
- The entire handling environment, including utensils and instruments, must be free of wax and grease so as not to affect the chemical adhesion between resins.
- Do not dispose of the residues in domestic sewers  
For dental use only.
- Keep away from light and heat.

### CHARACTERISTICS/BENEFITS:

- Ease of handling.
- Aesthetic and effective results in opacification of any metal structure.
- Independence in the polymerization mode.

### PRESENTATIONS:

Package with 1 unit of 10ml.

### COLOR:

Pink

### COMPOSITION:

Polymethylmethacrylate  
Ethyl Acetate  
Pigments.

### SHELF LIFE:

2 years as of date of manufacture.

### ANVISA Nº:

82426979010

### TECHNICAL OFFICER:

Jéssica Augusto  
CRQ IV-04270957

**ATTENTION:** Use our free support channels to obtain the correlation between the purchased product and the corresponding usage instruction version. To obtain, free of charge, these Instructions for Use in printed format, ask the customer service (SAC) on 0800 058 8888.

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