INDICAÇÃO:

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INDICATION:

Evolay is used for sculpting fixed prostheses and fixing their welding points, in addition to making shells for impressions, fixing implant transfers and all work where precision is necessary.

CONTRAINDICATIONS:

The use of this medical product must not be used in people with known skin sensitization (contact dermatitis) to polymerizable monomers, as they can cause allergic reactions to hypersensitive people.

INSTRUCIONS FOR USE:

To make the molding of the intra-radicular canal, it is important to use a guide pin made this resin. The canal should be insulated with insulator, put the powder and liquid each in a Dappen pot (indirect technique), with the use of a Marta brush, fill the canal with the resin, wetting the brush in liquid and later in the powder and introducing the guide pin. To make the sculptures and headgears, insulate the plaster model with Evofilm insulator, and then proceed with the indirect technique described above or with the direct technique.

DIRECT TECHNIQUE:

- 1- Put the liquid inside a Dappen pot and then the powder in the ratio of 2:1.
- 2- Spatulate as minimum as possible, cover the Dappen until the plastic phase is reached (phase when the resin breaks without forming threads).
- 3- When the plastic phase is reached the resin can be transferred:
 - to the plaster model (insulated with Evofilm)
 - to the patient's mouth.

WARNINGS/PRECAUTIONS:

- For dental use only.
- When there is a proven allergy to any of the components of the formula, the product should not be used.
- Use the product in a ventilated place using a spatula, gloves, mask and goggles.
- The handling of this product is not recommended for pregnant and lactating women.
- Keep the container closed.
- Do not reuse packaging.
- Correctly follow the instructions for use in order to maintain the properties and performance of the product set by the manufacturer.
- Follow the common procedures for microbiological control during manipulation.
- The entire handling environment, including utensils and instruments, must be free of wax and grease so that it does not affect the chemical adhesion between the resins.
- Do not dispose of waste in domestic sewage.
- Products that are not compatible with resin/monomer: alcohol, water, chloroform, acetone, vaseline and other solvents.
- Store away from light and heat.

CHARACTERISTICS/BENEFITS:

- High color stability.
- Ease of application.
- Selected raw materials, ensuring stability and product quality.
- Dust-dosing nozzle, for easy handling.
- Dimensional stability due to low contraction.

PRESENTATIONS:

For another presentations, contact us.

POWDER: 50g

LIQUID: 50ml

Shades:

Red.

COMPOSITION:

POWDER

Polymethylmethacrylate
Benzoyl Peroxide
Biocompatible pigments

LIQUID

Methylmethacrylate
DMT
Inhibitor

SHELF LIFE:

POWDER: 10 years as of the date of manufacture.

LIQUID: 2 years as of date of manufacture.

ANVISA N°:

POWDER: 82426979003

LIQUID: 82426979006

TECHNICAL OFFICER:

Jéssica Augusto
CRQ IV-04270957

ATTENTION: Use our free support channels to obtain the correlation between the purchased product and the corresponding usage instruction version. To obtain, free of charge, these Instructions for Use in printed format, ask the customer service (SAC) on 0800 058 8888.

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