INDICAÇÃO:

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INDICATION:

Evolut DLA suitable for full dentures, removable partial dentures, protocols over implants, restorations or provisional prostheses. Excellent abrasion resistance and durability convert it to the ideal choice for dental rehabilitations, providing not only functionality, but also comfort and aesthetics for your patients.

CONTRAINDICATIONS:

Unknown adverse side effects when product is used according to usual procedures.

WARNINGS AND PRECAUTIONS:

Keep away from light and heat.
For dental use only.
Use the Evolut DLA shade guide and articulation table.

INSTRUCTIONS FOR USE:

Follow the usual procedures for selection, assembly, articulation, acrylization, microbiological control and maintenance of denture, as well as for individual protection.

For a better teeth adhesion to the polymer prosthesis base, remove the brightness from the tooth base using an abrasive tool. After flask insulation and before acrylic application, clean the teeth base using a wax remover product.

Recommendations to the patient: To increase teeth durability, daily hygiene is recommended without using any aggressive substances, like acids or alkalis.

CHARACTERISTICS / BENEFITS:

- Evolut DLA (Double Linked Active) features an innovative organic additive capable of providing exceptional benefits through dual crosslink reinforcement, increasing wear and impact resistance. This advanced system extends the prosthesis life and provides greater patient comfort.
- Pressed in a three layering process, providing a natural and realistic appearance.
- Anatomy designed to provide teeth with natural characteristics, featuring specific textures that enhance translucency and realism in the prosthesis.
- High-precision production process using stainless steel matrices developed by CAD/CAM technology, ensuring greater symmetry and details richness.
- Selected raw materials, compatible with prosthesis base, ensuring better adhesion and integration.
- Wide range of colors and moulds.
- Enhanced layer color stability, guaranteeing long-lasting and natural aesthetics.
- Characteristically bluish incisal edges, providing excellent aesthetic results.
- Biocompatible pigments, safe and suitable for dental use.

- Natural fluorescence, offering an authentic look under any different lighting conditions.

- Compliant to ISO 22112, ensuring quality and safety.

MODELS:

Upper Anterior:

O43, O47, R45, R46, R47, R49, S43, S44, T44, T45 e T46.

Lower Anterior:

L33, L35, L36 e L38.

Upper and Lower Posterior:

30, 32 e 34.

SHADE:

A1, A2, A3, A3,5, A4, B1, B3, C2, C3, D3, BL2, BL3.

COMPOSITION:

Polymethylmethacrylate
Ethylene Glycol Dimethacrylate
Fluorescent
Biocompatible Pigments
Organic additive.

SHELF LIFE:

10 years as of the date of manufacture.

ANVISA Nº: 82426979002

TECHNICAL RESPONSIBLE:

Jéssica Augusto.
CRQ IV-04270957

ARTICULATION CHART	
ANTERIOR	POSTERIOR
T44 - L33 - L35	30 - 32
T45 - L33 - L35 - L36	32 - 34
T46 - L35 - L36 - L38	32 - 34
O43 - L33 - L35 - L36	30 - 32
O47 - L38 - L39	34
R45 - L33 - L35	32
R46 - L33 - L35 - L36	32 - 34
R47 - L38	34
R49 - L38	34
S43 - L33 - L35 - L36	30 - 32
S44 - L33 - L35 - L36 - L38	30 - 32 - 34

ATTENTION: Use our free support channels to obtain the correlation between the purchased product and the corresponding usage instruction version. To obtain, free of charge, these Instructions for Use in printed format, ask the customer service (SAC) on 0800 058 8888.

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