INDICAÇÃO:

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INDICATION:

Reference wall for framework planning, matrix for making provisional prostheses, wall for including complete dentures, model duplications in general, walls for pressing stock teeth in complete dentures and PPR.

WARNINGS/PRECAUTIONS:

- Keep at room temperature;
- Do not use latex gloves;
- No special protective measures are required;
- Store away from light and heat;
- Do not dispose of waste in drains, watercourses or household waste.

INSTRUCTIONS FOR USE:

Use equal amounts of both components, Mass A and Mass B, with the measuring spoons provided. Respect the 1:1 dosage. Do not change the measuring spoons or lids. Mix the mass with your fingers to obtain a uniform color in about 1 minute and 20 seconds. The product is then ready to use for about 3 minutes and 50 seconds.

FEATURES AND BENEFITS:

- Resistance to heat over 215° C
- Excellent definition of details
- Guaranteed size stability
- Easy to use
- Biocompatible;
- Shore A 90 hardness.

PRESENTATIONS:

500g Mass A

500g Mass B

Note: Contains measuring spoons (1 White, 1 Blue)

COLORS:

Evosil A-90 Mass A: Gray

Evosil A-90 Mass B: Blue

COMPOSITION:

Polydimethylsiloxane, Mineral Fillers and Additives.

VALIDITY:

24 months.

Product exempt from registration by the Ministry of Health.

TECHNICAL MANAGER:

Jéssica Augusto.
CRQ IV-04270957

ATTENTION: Use our free support channels to obtain the correlation between the purchased product and the corresponding usage instruction version. To obtain, free of charge, these Instructions for Use in printed format, ask the customer service (SAC) on 0800 058 8888.

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