**INDICAÇÃO:**

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\*Consulte disponibilidade.

**INDICATION:**

For use in permanent or removable, total or partial dentures, overdentures, crowns or temporary bridges.

**CONTRAINDICATIONS:**

There are no known adverse side effects, when the product is used following the usual procedures.

**WARNINGS AND PRECAUTIONS:**

Keep away from light and heat.  
For dental use only.  
Use the Evotri shade guide and articulation table.

**INSTRUCTIONS FOR USE:**

Follow the usual procedures for selection, assembly, articulation, acrylization, microbiological control and maintenance of the denture, as well as for individual protection.

For better adhesion of the teeth to the polymer base of the prosthesis, remove the brightness from the base of the tooth with an abrasive tool. After insulation of the flask and before application of the resin, clean the base of the teeth with wax remover.

Recommendations for the patient: To increase durability of the teeth, daily hygiene is recommended without use of aggressive substances, like acids or alkalis.

**CHARACTERISTICS / BENEFITS:**

- Pressing of 3 layers, providing a natural appearance.
- Produced through CAD-CAM steel moulds, with great symmetry and details.
- European model set compatible with diverse age group and with better anatomical adjustment.
- Manufacture with selected raw materials, compatible to those of the denture base, improving it's adhesion.
- High stability of colors, with more natural colors.
- Biocompatible pigments
- With fluorescence, providing natural look and a realistic appearance.
- Complies with ISO 22112
- It contains an organic additive that increases abrasion and impact resistance, improving the prosthesis' life span and the patient's comfort

**MODELS:****Upper Anterior:**

E2, E3, E4, F4, G3, H3, H4, L2, L4, O32, O36, R13, R14, R17, T25, T4B, O31, R11, R12, R66, T21, T22, T44.

**Lower Anterior:**

K6, K10, K12, L1, L3, L5, L7 e L10.

**Upper and Lower Posterior:**

M2, M3, M4, M5 e M6.

**SHADE:**

A1, A2, A3, A3,5, A4, B1, B3, C2, C3, D3, BL2, BL3, 1A, 1C, 1D, 1E, 2A, 2B, 3E, 4A e 4B.

**COMPOSITION:**

Polymethylmethacrylate  
Ethylene Glycol Dimethacrylate  
Fluorescent  
Biocompatible Pigments  
Organic additive

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**ATTENTION:** Use our free support channels to obtain the correlation between the purchased product and the corresponding usage instruction version. To obtain, free of charge, these Instructions for Use in printed format, ask the customer service (SAC) on 0800 058 8888.

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