INDICAÇÃO:

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INDICATION:

Self-curing acrylic resin for orthodontic work. Ortoden is indicated for manipulation technique. Replacing part of the masticatory system of the human body.

CONTRAINDICATIONS:

This medical product must not be used in people with known skin sensitization (contact dermatitis) to polymerizable monomers, as they may cause allergic reactions to hypersensitive people.

WARNINGS/PRECAUTIONS:

- For dental use only.
- Liquid: Flammable and non-toxic material (Flash-point: 10°C/50°F).
- Resin: May form flammable air/dust mixtures.
- As with all acrylic resins, it is recommended to handle the product in ventilated areas, using nitrile gloves and protective goggles; however, handling is not recommended for pregnant women, nursing mothers and people with recognized allergies to this type of product.
- May cause skin irritation in susceptible people.
- Keep away from light and heat. Keep in a dry and cool place.
- Do not dispose of waste into sewers and water courses or in domestic waste.
- Products that are not compatible with the resin/monomer: alcohol, water, chloroform, acetone, petroleum jelly and other solvents.
- The bases of the synthetic teeth must be ready in accordance with the Instructions for Use for the correct support of the prosthesis.
- We do not recommend the use of wax solvents. The residual may cause a lower connection of the teeth with the acrylic base.
- Recommended mechanical polishing for all types of resin.

INSTRUCTIONS FOR USE:

Shake the bottle before using it.

The basic and essential prerequisites for making orthodontic appliances must be respected. Before starting any work, the parts to be used must be completely clean and free of traces of wax or fat, so that the chemical union between the resins is perfect.

In the manipulated technique, use ORTODEN resin, placing the necessary amount of liquid in the glass, silicone or Dappen pot, according to the amount of acrylic. Over the liquid, put the resin in the proportion powder/liquid of 2 measures of powder for 1 measure of liquid (14g of powder for 7ml of liquid) until the two are mixed without any excess of powder or liquid; if there is any, discard it. Homogenize carefully with the spatula, avoiding the incorporation of air bubbles by applying the paste to the model.

After application, eliminate the excess material and, still in the plastic phase, eliminate the bubbles through a hydrostatic press in a polymerizer (put the necessary amount of water, apply 20 pounds of

compressed air and let the polymerization complete for 20 Minutes). After complete polymerization start the finishing with burs and cutters, then with abrasive stones and finish with fine texture sandpaper. Polish with a soft brush, using FINISHING AND POLISHING ABRASIVE.

The ORTODEN resin is specially elaborated, in several colors, to broaden its field of use by professionals in the area, in addition to motivating its use by patients. The biocompatible pigments have high stability. The resins can be mixed to achieve new shades. High color stability and abrasion resistance.

EVODEN recommends only mechanical polishing for this product.

CHARACTERISTIC/BENEFITS:

- The rigorous selection of raw materials, receiving tests combined with successive tests during all manufacturing cycles, guarantee the specified results in the final product.
- Supports the most varied polymerization methods for self-curing resins, professionally known.
- Liquid with Crosslink ensuring greater resistance to the product.
- Easy handling and application for the execution of the work.
- Wide range of colors, ensuring the customization of your work.

PRESENTATIONS:

POWDER: 50g and 120g.

LIQUID: 50ml.

SHADES:

Blue, Colorless, Pink Electric, Pink, Green, Lime Green, Red, and Purple.

FOR ANOTHER PRESENTATIONS, CONTACT US.

COMPOSITION:**POWDER**

Polymethylmethacrylate
Benzoyl Peroxide
Biocompatible pigments

LIQUID

Methyl methacrylate monomer

DMT

EDMA (Crosslink)

Inhibitor

STORAGE

Keep away from heat and humidity. Keep in closed container after use.

SHELF-LIFE:

POWDER: 10 years as of the date of manufacture.

LIQUID: 2 years as of the date of manufacture.

ANVISA N°:

POWDER: 82426979003

LIQUID: 82426979007

TECHNICAL RESPONSIBLE:

Jéssica Augusto

CRQ IV-04270957

ATTENTION: Use our free service channels to obtain the correlation between the supplied product and the version of the corresponding Instructions for Use. To obtain, free of charge, these Instructions for Use in printed form, please ask Customer Service at 0800 058 8888

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